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TECHNICAL DOCUMENTATION

INTENDED USE

Telo Bagnoletto is an aid that is used in the field of assistance to bedridden people (disabled, elderly convalescents). It is designed for use in hospitals, nursing homes, home care operators and is intended for non-self-sufficient users.

The device is a technical aid (as defined in UNI EN 12182:2012): Instrument/equipment (for disabled persons) or technical system which, according to the manufacturer, is to be used for the prevention, treatment or alleviation of injury, impairment, disability or handicap or for the compensation thereof.

BAGNOLETTO is NOT intended for lifting either people or the contents of the washing liquid.

Make sure the surface on which the sheet is to be placed is STABLE and large enough for the sheet to be fully and properly supported.

Check the integrity of the sheet, any breaks, cracks, leaks, etc.

In case of doubt about the integrity of the product, contact the manufacturer by sending an email to: info@allmobility.it

SYMBOLS

Read the Instructions for Use carefully before use.

This symbol indicates not to throw in the common garbage.

Not recyclable.
Dispose of via undifferentiated collection

Class I Medical Device

Lot Number

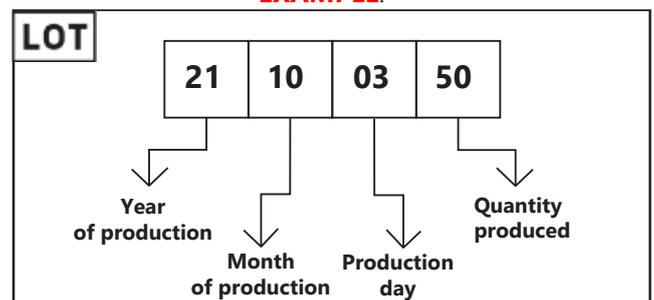
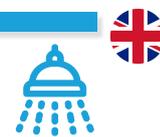
Allmobility Trading s.r.l
Via G.Balla, 4 - Reggio Emilia (RE)
0522941078 - Info@allmobility.it

The expected life of the product is 5 years from the date of production which can be seen from the lot number.

How to read the Lot number correctly?

EXAMPLE:

TECHNICAL CHARACTERISTICS

Bagnoletto is a class 1 medical device measuring 85 x 195 cm, h 20 cm with polyester fabric, PVC coating and lacquering on both sides, embossed on top, flame retardant secondo DIN 75 200

TECHNICAL CHARACTERISTICS OF PRIMARY COMPONENT (BLUE CLOTH)

Yarn: Polyester (pet)

Plot: 1100 Dtex

Thickness: 3 mm

Fabric weigh: 650 gr/m²

Heat resistance: + 70°

Cold resistance: -30°

Fire resistance: burning speed < 100 mm / min (DIN 75200)

Resistance to damage by lexing: > 100.000 (DIN 53359)

METHOD OF USE

1 | Lay the bedridden person on his side, place Bagnoletto next to him and unfold it completely.

2 | Place Bagnoletto as far as possible under the person.

3 | Turn the person onto his other side.

4 | Place the person in a supine position.

5 | Lift up all the sides and join them at the corners with the clips.

6 | Insert the flexible hoses provided in the appropriate water drains.

7 | Place a container on the floor to collect the waste water.

8 | Take the pressure tank filled with water and pump until the desired pressure is reached.

9 | Start to wash the bedridden person.

10 | After drying the person, proceed to disassemble the flexible hoses.

11 | Lay the person on his side and fold down the sides of the Bagnoletto.

12 | Turn the person onto his other side, passing over the edge of Bagnoletto.

13 | Retrieve Bagnoletto and fold it.

INSTALLATION OF WATER DRAINS AND FLEXIBLE HOSES

1 | Carefully screw on the bottom part of the water drain.

2 | Connect the flexible hose to the water drain.

HOW TO STORE BAGNOLETTO

HOW TO STORE BAGNOLETTO

1 | Unfold Bagnoletto completely.

2 | Fold all the sides inwards, starting from the short end.

3 | Fold one third of Bagnoletto inwards.

4 | Perform the same procedure on the opposite side, thus forming a rectangle. Turn Bagnoletto to the opposite side and fold a third of it inwards.

5 | Fold a third of Bagnoletto inwards again.

6 | Perform the same procedure on the opposite side.

Accessory: tank
Hose and shower assembly

TECHNICAL CHARACTERISTICS

Maximum capacity: 10 litres

Maximum temperature: from 1°C to 40°C

Maximum pressure: 3 bar

1 | Pull the knob on the pressure relief valve to ensure that the tank is not under pressure and the valve is working properly.

4 | Check that the inside edge of the tank and the gasket of the pumping element are clean.

5 | Reinsert the pumping element in its seat.

3 | Pour in water at the desired temperature or the previously prepared and filtered solution up to 2/3 of the total tank capacity.

6 | Screw the pumping element back on firmly in a clockwise direction and put it under pressure.

INSTALLING FLEXIBLE HOSE AND HAND SHOWER

1 | Pass the hose through the hole at the bottom of the pump.

2 | Carefully screw the hose onto the pump, remembering to insert the gasket supplied.

3 | Screw the hand shower onto the supplied hose with the appropriate gasket.

USING PROCEDURE

1. Release the handle from the clips by turning it slightly counter-clockwise and pump 4-5 times alternately.
2. After a few moments, pull the knob of the pressure relief valve and check that air is escaping.
3. Continue pumping until the desired pressure is reached and in any case no more than 3 bar (approximate calibration level of the relief valve)
4. To spray water, press the handle. Adjust the jet according to the treatment to be carried out by turning the end of the spray unit.
5. Choose the desired type of spray.
6. When the jet weakens it is necessary to restore the pressure inside the tank.

AFTER USE

1. Keeping the pump in an upright position, pull the knob of the pressure relief valve.
2. Completely discharge the residual pressure from the tank.
3. Unscrew and remove the pumping element.
4. Pour all the remaining liquid into a container and rinse the inside of the tank and hose thoroughly with clean water, spraying water through the hand shower.
5. Open the handle and check periodically that the filter is intact and clean.
6. Store the pump, upside down and with the pumping element slightly unscrewed, in a clean and dry place.

WARNINGS

This pump is intended for the delivery of aqueous solutions with a density below 1.1 and a temperature not exceeding 40°C. It is important to strictly follow the instructions given by the manufacturer of the dispensed substance.

Do not dispense products for which the manufacturer's technical data are not available.

Under no circumstances may the pump be used with solvents, acids, flammable and explosive liquids, corrosive liquids, disinfectants and impregnating agents.

Allmobility Trading srl
via G. Balla, 4
42124 Reggio Emilia
Tel. 0522 941078
Mob. 388 6389624
Fax 0522 946625
info@allmobility.it
www.allmobility.it